[EN] Consumer Information

An innovative gel for the management of mucosal conditions.

StrataMGT is a semi-permeable, non-resorbable and self-drying gel.

When used as directed, StrataMGT lightly bonds to form a protective layer that is gas permeable and waterproof which hydrates and protects mucosal tissue.

StrataMGT may be directly applied to dry, wet, cracked and sensitive mucosal tissue.

StrataMGT has no measurable pH and contains no steroids, hormones, alcohol, parabens or fragrances, and is suitable for long-term use.

StrataMGT gel is bacteriostatic and inert.

How StrataMGT works

StrataMGT promotes a moist wound healing environment.

This moist wound healing environment supports the body's natural wound healing process which leads to faster healing, significantly reduces the acute inflammatory response and temporarily restores the barrier function of the mucosa.

StrataMGT hydrates and protects affected mucosa.

When used inside the vagina or rectum, StrataMGT will not be absorbed systemically.

StrataMGT will not build up inside the orifice, it will slowly be excreted with body fluids.

Intended use

StrataMGT is used to restore the barrier function and moisture balance of the mucosal epidermis.

The moisture barrier that StrataMGT creates, supports the body's normal healing process, this in turn leads to relief from symptoms such as itchiness, tenderness, dryness, burning sensation, painful intercourse (dyspareunia), painful urination (dysuria) and rectal or defecation pain.

The restoration of the barrier function also improves erythema, mucosal tissue thinning, erosions, fissures, ulcerations, scarring/adhesions and swelling.

StrataMGT was developed for use on all types of mucosal conditions and post-procedures care where the epidermis of the vaginal mucosa is damaged or wounded including:

- Genitourinary syndrome of menopause, GSM (vaginal atrophy / atrophic vaginitis)
- Vulvovaginitis (including pre-pubertal)
- Vulvodinia
- Lichen sclerosus
- Lichen planus/simplex
- Post vaginal rejuvenation
- Vaginal grazing
- Episiotomy
- Incontinence related dermatitis
- Diaper rash
- Hemorrhoids and anal fissures
- Allergic and irritant dermatitis


StrataMGT may also be used in conjunction with other adjunctive treatments to improve overall results.

StrataMGT is suitable for infants, children and people with sensitive skin, during pregnancy and while breastfeeding.

StrataMGT is intended for single patient use.

StrataMGT does not interfere with sexual intercourse.


Directions for use

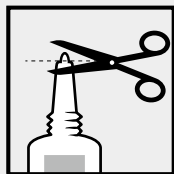On first use of 0.17 oz (5 g) and 1 oz (28.4 g) tubes remove the cap, cut the nozzle tip of the tube and close with the cap after use.

Wash your hands before applying StrataMGT.

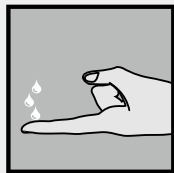When used vaginally, apply 3 to 5 drops of StrataMGT on the tip of your finger

Distribute evenly externally and/or internally.

Drying is not necessary. For best results StrataMGT should be maintained in continuous contact with the affected area (24 hours a day / 7 days a week).

StrataMGT should be applied as needed or as advised by your physician. StrataMGT should be applied after each urination, bowel movement or wash. StrataMGT may be re-applied more often if needed.

StrataMGT does not need to be rubbed in or massaged, as it does not penetrate through mucosa or skin and will not enhance its effect.

Recommended duration of treatment

StrataMGT should be applied for a minimum of 30 days.

StrataMGT is intended for long-term use to maintain the health of the mucosa.

How much StrataMGT do I need?

StrataMGT gel is an advanced formulation that requires substantially less product per application than typical creams or gels.

- One **StrataMGT 0.17 oz (5 g)** tube contains enough gel for approximately 2 weeks of use 1–2 times per day.
- One **StrataMGT 1 oz (28.4 g)** tube contains enough gel for approximately 4 weeks of use 2–3 times per day.

Caution

- StrataMGT should not be placed in contact with the eyes.
- StrataMGT may also be used in conjunction with other treatments when advised by your physician.
- StrataMGT may reduce the effectiveness of contraception such as spermicides, condoms or vaginal devices. Please use other precautions during this time.
- StrataMGT may increase lubrication and therefore should not be used in conjunction with condoms for the prevention of sexually transmitted diseases (STDs).
- StrataMGT may stain clothing or sheets. If staining occurs, dry cleaning should be able to remove it without damaging the fabric.
- For correct storage please reclose the tube tightly with the cap.
- For chronic conditions regular surveillance of your physician is recommended.
- Should you show signs of infection or failure to heal, consult your physician.
- If irritation occurs, discontinue use and consult your physician.
- Keep out of the reach of children.
- Do not use after the expiration (EXP) date printed on the tube. The expiration (EXP) date does not change once the tube has been opened.
- Do not use if the tube is damaged.
- StrataMGT does not require special disposal methods. Please follow your local disposal regulations.

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Stratpharma AG